Paladur®

DE

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GB

Instructions for use

FR

Mode d'emploi

ES

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IT

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NL

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DK

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GR

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KULZER
MITSUI CHEMICALS GROUP

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

Explanation of symbols on labelling	
	Manufacturer
	Batch code
	Use-by date
	Date of manufacture
	Consult instructions for use
	Storage temperature
	Unique Device Identification

	Pulver Powder Poudre Polvora Polvere Poeder Pulver pulvpráš Prášok
	Flüssig Liquid Liquido Liquido Liquido Vloeibaar Flydende povodná Kvapaliné

	Medizinprodukt Medical device Dispositif médical Produto sanitário Dispositivo medico Medicinsk hulpmiddel Medicinsk utrust Iatronizologiki proutia Zdravotnicka pomôcka
--	--

CE 0197

GB Instructions for use Paladur®

Medical Device – for use by dental health care professionals only.

Intended purpose

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant. Kulzer medical devices enable the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

Product description

Paladur is an autopolymerising prosthetic resin for all the indications listed below. It consists of a powder component in a range of colours (pink, pink veined, R50 veined, clear) and liquid. Prosthetic resin for casting fixed and removable prosthetic devices that cures in thin layers on contact with air without technical effects.

Composition

Cold-curing polymer, consisting of powder and liquid. Contains benzoyl peroxide.

Powder component: acrylic polymer; 5-n-butyl barbituric acid; dibenzoyl peroxide

Liquid components: methyl methacrylate; 1,4-butanediol dimethacrylate

Indications

For fixed and removable prosthetic devices.

Can be used for:

- Restoring function
- Extension
- Additions
- Repairs
- Indirect partial and total relines

Contraindications

The intraoral use of uncured Paladur dough is contraindicated. In case of known or suspected allergy to components of the product, its use is contraindicated. Do not use in case of known or suspected allergy against (methyl) acrylic compounds or benzoyl peroxide.

Potential side effects

Hypersensitivities to the product or its components can not be excluded in individual cases. In case of suspicion please contact the manufacturer for ingredients information. Hypersensitivity reactions (allergies), local sensory disturbances, taste disturbances and irritation of the oral mucosa have been described in isolated cases. After processing, all MMA/PMMA-based denture materials always contain a permitted, small proportion of residual monomer (<4.5 %). For further risk reduction of intolerance reactions, the denture has to be stored in tepid water for at least 12 hours before placement.

Processing

Dental stone type 3 is recommended for the production of models. When performing extensions or repairs, use hot water. Remove residual wax carefully from the resin surfaces to be bound. Roughen a wide area on the resin parts to be bound (e.g., when repairing a break); no mechanical retention is required. The wet plaster model is insulated twice using thin layers of Aislar® (see Aislar® Instructions for Use). This is the only way to ensure perfect surface quality between Paladur and the model. To improve the bond between Paladur and prosthetic resins or the prefabricated teeth, the contact surfaces are roughened with a coarse diamond bur (remove grinding dust) and wetted with Palabond®. Apply Palabond® twice using a brush (without metal ferrule) and allow to set for 30 seconds each time. After the second application the adhesive remains active for 10 minutes.

Amounts required

The recommended mixing ratio is 10 g powder : 6 ml liquid. Introduce the liquid into the mixing beaker, add the appropriate quantity of powder rapidly and stir for up to 15 seconds to make homogeneous paste. To remove air bubbles, swirl the mixing beaker. Working time depends on ambient temperature. The drippy liquid paste can be worked immediately. Apply Paladur evenly and once the plastic consistency has been attained, do not mix or model further.

Polymerisation

The curing time at room temperature 23°C (73°F) is approx. 9–12 minutes. The optimal polymerisation without formation of bubbles is achieved with the Palamat® Premium / Palamat® elite for 10 minutes, at a water temperature of 55°C (131°F) and a pressure of 2 bar. When working without a pressure polymerisation unit, apply larger amounts of Paladur in layers. Each layer must be allowed to cure and cool completely before new paste is applied.

Trimming and polishing

After cooling and re-occlusion the prosthesis is carefully removed from the model and trimmed with cross-toothed cutters. Before polishing and grinding, the prosthesis should be made smooth with sandpaper of decreasing grain size.

Repair

All cold-curing Pala prosthetic resins from Kulzer are compatible with one another and can be used for restoration/addition and repairs. The heat-curing Pala prosthetic resins from Kulzer can also be restored, added to and repaired with all cold-curing Pala prosthetic resins.

Storage advice

Do not store above 25°C (77°F). Do not use the material after the expiration date. Avoid direct sunlight. When product has been removed, reseal carefully.

General information

Please pay attention to the following safety instructions and all processing instructions in other sections.

This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

Warnings and safety instructions

May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Highly flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources – no smoking. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Toxic to aquatic life with long lasting effects. Flash point: 10°C.

For SSCP please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact scsp-service@kulzer-dental.com. To find the SSCP in the EUADAMED use the UDI code attached at the packaging of the product.

Application notes

Examine the integrity of the primary packaging before first use. Damaged products must not be used. When positioning teeth in wax, do not use alcohol based separating agents (separation of gypsum and wax), as this may cause white discoloration on the basal surface of the prosthesis.

Complaint notice

Please quote batch number (LOT) and article number (REF) or UDI in all correspondence about the product.

According to EU Medical Device Regulation users / patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Disposal information

Recommendation: Dispose of in accordance with official waste disposal rules. Do not dispose of partially empty kit packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Safety data sheets and more information are available at our website www.kulzer.com

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® = registered trademark of Kulzer GmbH
® = registrovaná ochranná známka spoločnosti Kulzer GmbH